PlastiPad®

Infant

Reusable Hyper-Hypothermia Blanket Thermal Regulating System

INDICATIONS FOR USE

The PlastiPad® reusable hyper-hypothermia blanket is intended to be used by trained healthcare professionals as part of CSZ's thermal regulating systems, specifically the Blanketrol® II, Blanketrol® III, Hemotherm®, Norm-O-Temp® or Micro-Temp LT® to lower or to raise a patient's temperature and/or maintain a desired patient temperature through conductive heat transfer.

CONTRAINdications

None known.

SET UP

- Unfold blanket on bed, stretcher, or operating room (OR) table with hose routed without kinks towards the unit. Check blanket before each use for product integrity.
- Check the equipment reservoir to ensure that it is full of distilled water.
- Connect male coupling and female coupling from the Cat#286 hose to unit. Make sure they "click" as they lock together.
- Attach blanket to the Cat#286 hose by inserting the blanket connectors into the hose connectors. Connectors should "click" as they lock together.
- If using the blanket with the Micro-Temp LT®, connect the blanket directly to the hose attached to the device. Connectors should "click" as they lock together.
- Turn unit ON and select desired temperature setting.
- Place a single sheet between the patient's skin and blanket. The blanket may be filled either before or after the patient is placed on blanket. Blanket may be under or over the patient as desired.
- After filling and positioning, make sure blanket and hose are free of kinks that may restrict water flow.

OPERATING INSTRUCTIONS


- Follow the CSZ thermal regulating system operation manual for operating instructions.
- To discontinue therapy: turn the regulating system unit off. The water in the blanket will automatically drain back into the unit. Allow it to fully drain before disconnecting the blanket from unit or removing from bed, stretcher, or table.

CLEANING

- Clean blanket with a mild soap and water solution. Disinfect according to hospital/institution protocol. CSZ does not recommend sterilization. Do not autoclave.

STORAGE

- To store blanket, gently fold the corner containing the tubing towards the center of the blanket. Roll or fold blanket loosely avoiding sharp creases.
- Store blanket in cool, dry environment, free from items that may puncture blanket.

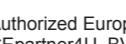Authorized European Representative
CEpartner4U, BV
Esooornlaan 13
3951 DB Maarn
The Netherlands
www.CEpartner4U.com

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